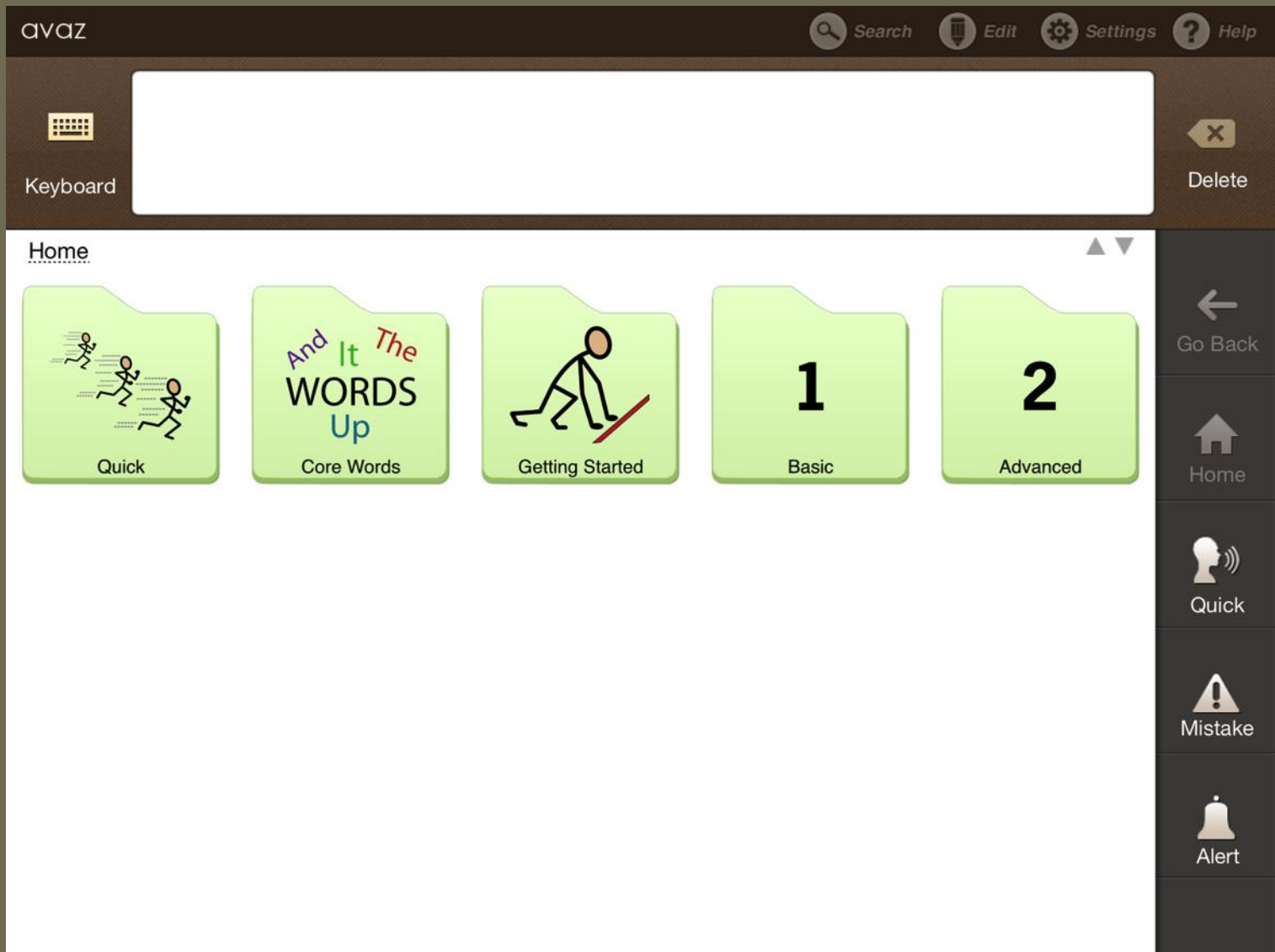
Avaz Manuale

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avaz Search Edit Settings Help

Keyboard   I  like  to  play  drums  Delete

Home → Advanced

 I	 you	 that	 do	 go
 is	 like	 want	 to	 not
 food	 indoor	 places	 people	 questions

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The screenshot shows the AVAZ app interface. At the top, there is a navigation bar with 'avAZ' on the left and 'Search', 'Edit', 'Settings', and 'Help' on the right. Below this is a 'Keyboard' section. A red box highlights a sentence construction area containing four icons: a stick figure, a stick figure pointing to a red square, a red arrow, and a stick figure playing a guitar. Below these icons are the words 'I', 'want', 'to', and 'play'. To the right of this area is a 'Delete' button with an 'X' icon. Below the red box, there is a breadcrumb trail: 'Home → Advanced → indoor'. The main area contains a grid of 15 icons, each with a label: 'more', 'my turn', 'your turn', 'all done', 'something else', 'can I', 'play with toys', 'watch TV', 'listen to music', 'read books', 'build stuff', 'play board games', 'play indoor games', 'do sewing', and 'computer and devices'. On the right side, there is a vertical navigation bar with buttons: 'Go Back', 'Home', 'Quick', 'Mistake', and 'Alert'.

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avaz Search Edit Settings Help

Keyboard I want to drink orange juice Delete

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
Vocabolario esaustivo

The screenshot shows the Avaz app interface. At the top, there is a navigation bar with the app name 'avaz' on the left and icons for 'Search', 'Edit', 'Settings', and 'Help' on the right. Below the navigation bar is a large white text input field with a keyboard icon on the left and a 'Delete' button on the right. Below the input field, there are two tabs: 'Home' and 'Advanced'. The main content area displays a grid of eight green folder icons, each representing a different category of vocabulary. The folders are: 'questions' (two red question marks), 'time & calendar' (a calendar grid and a clock), 'actions' (stick figures performing various actions like riding a bicycle and using a hammer), 'position' (a red cube with various prepositions like 'Over', 'On', 'Upon', 'Within', 'In', 'Into', 'Beside', 'Near', 'By', 'Below', 'Beneath', 'Under'), 'describe' (a stick figure pointing to an apple with speech bubbles), 'categories' (a grid of icons representing different categories), 'chat' (two stick figures talking), and 'my topics' (a stack of folders with a green arrow pointing to the top one). The 'my topics' folder is highlighted with a red border. On the right side of the screen, there is a vertical sidebar with navigation buttons: 'Go Back' (left arrow), 'Home' (house icon), 'Quick' (head with sound waves), 'Mistake' (warning triangle), and 'Alert' (bell icon).

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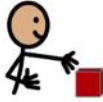
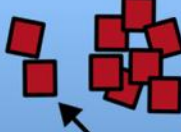
avaz Search Edit Settings Help

Keyboard   I  want  to  drink  orange juice  Delete

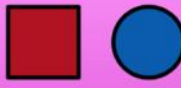
Home → Advanced → food

 I	 drink	 eat	 like	 to
 full	 hot	 hungry	 some	 thirsty
 warm	 all done	 food	 something else	 can I have some more?

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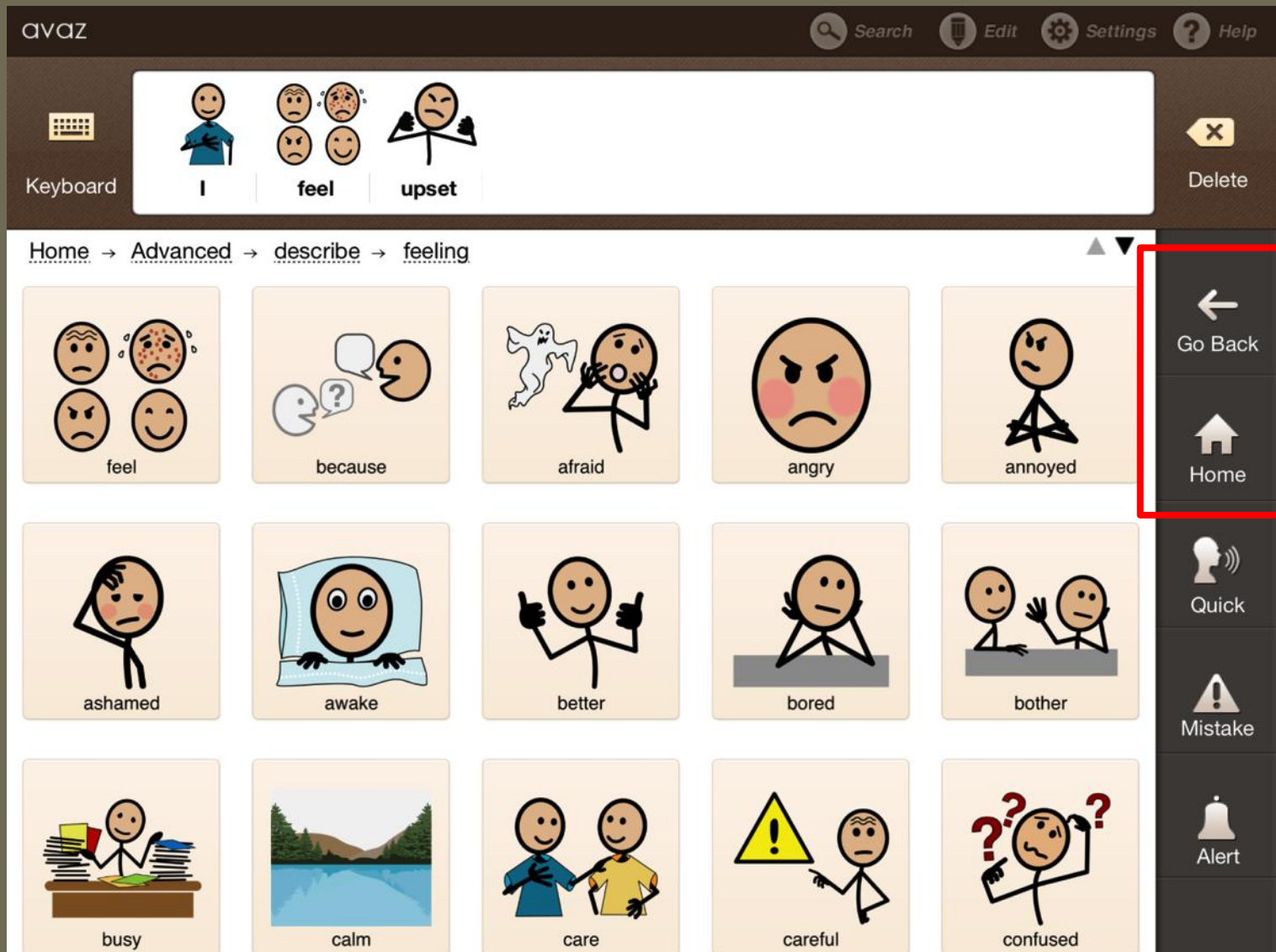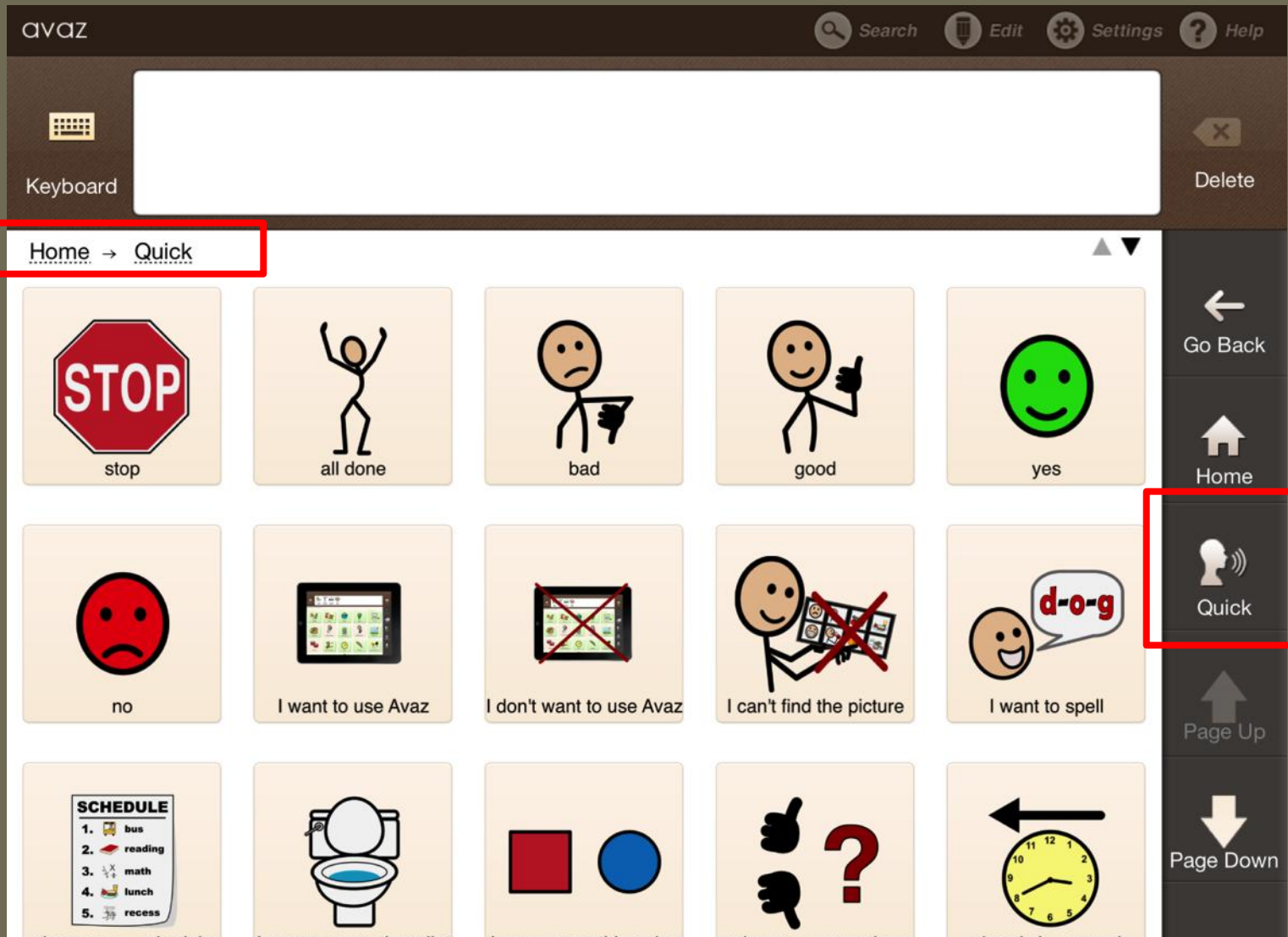
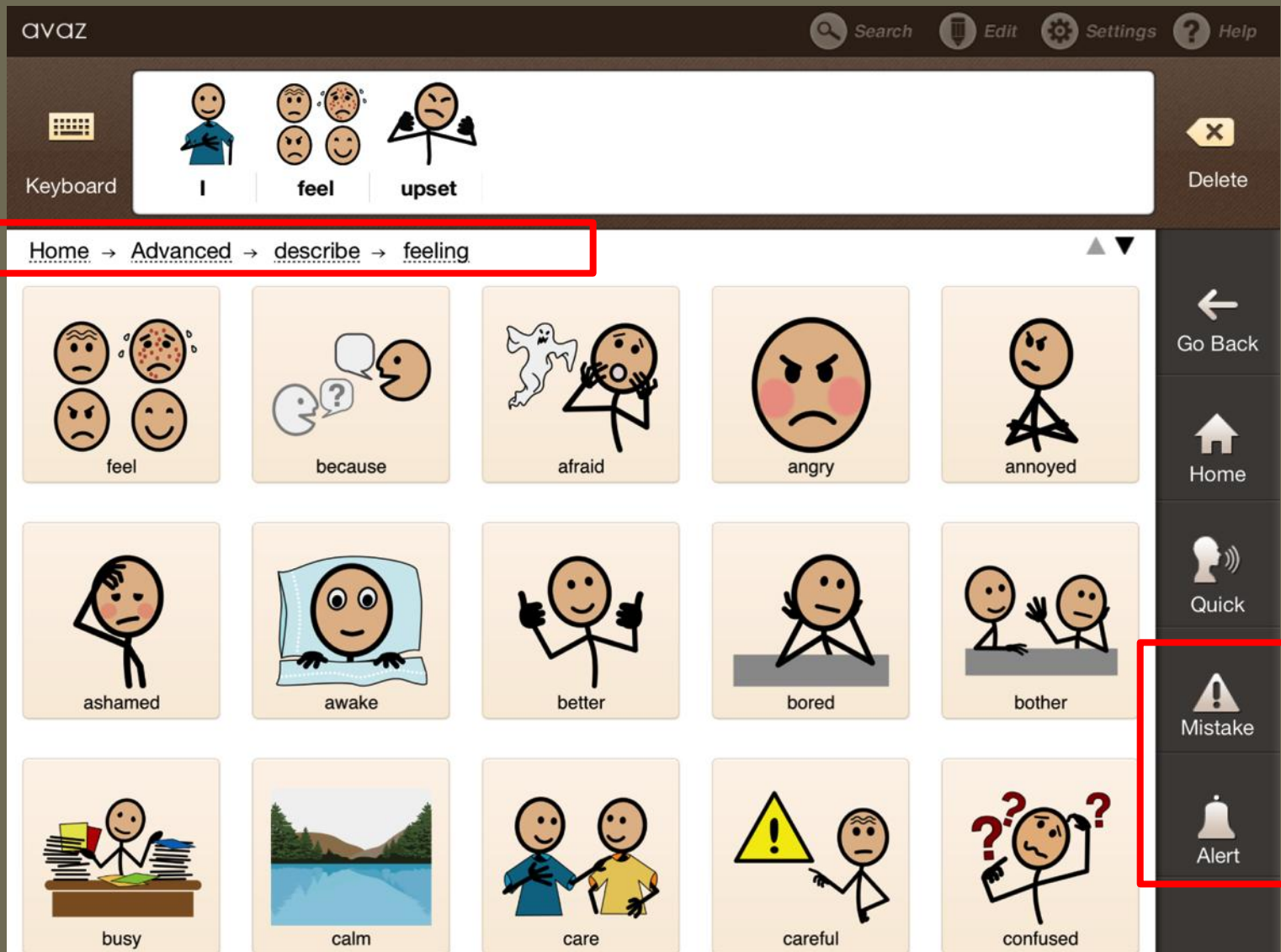
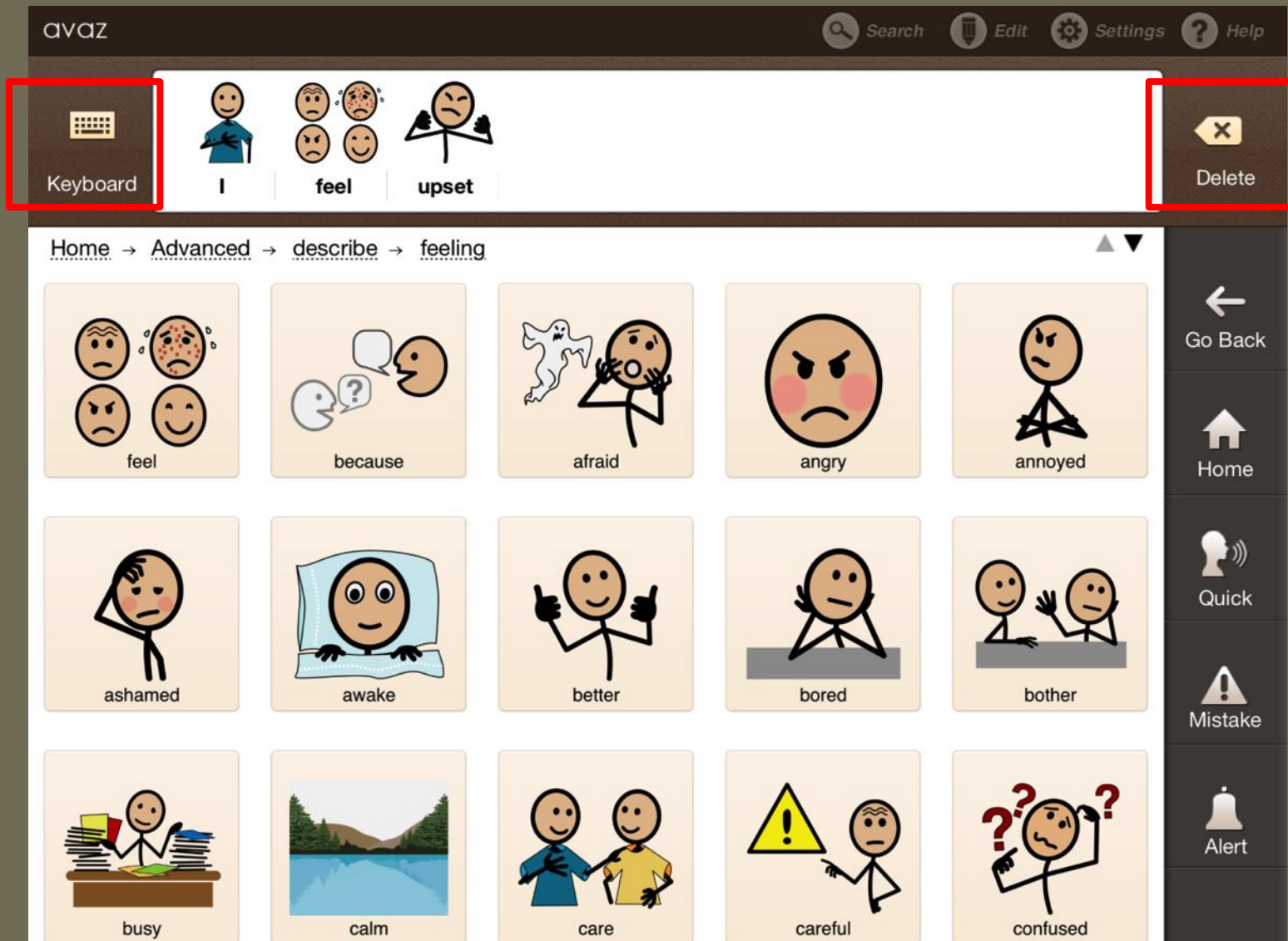
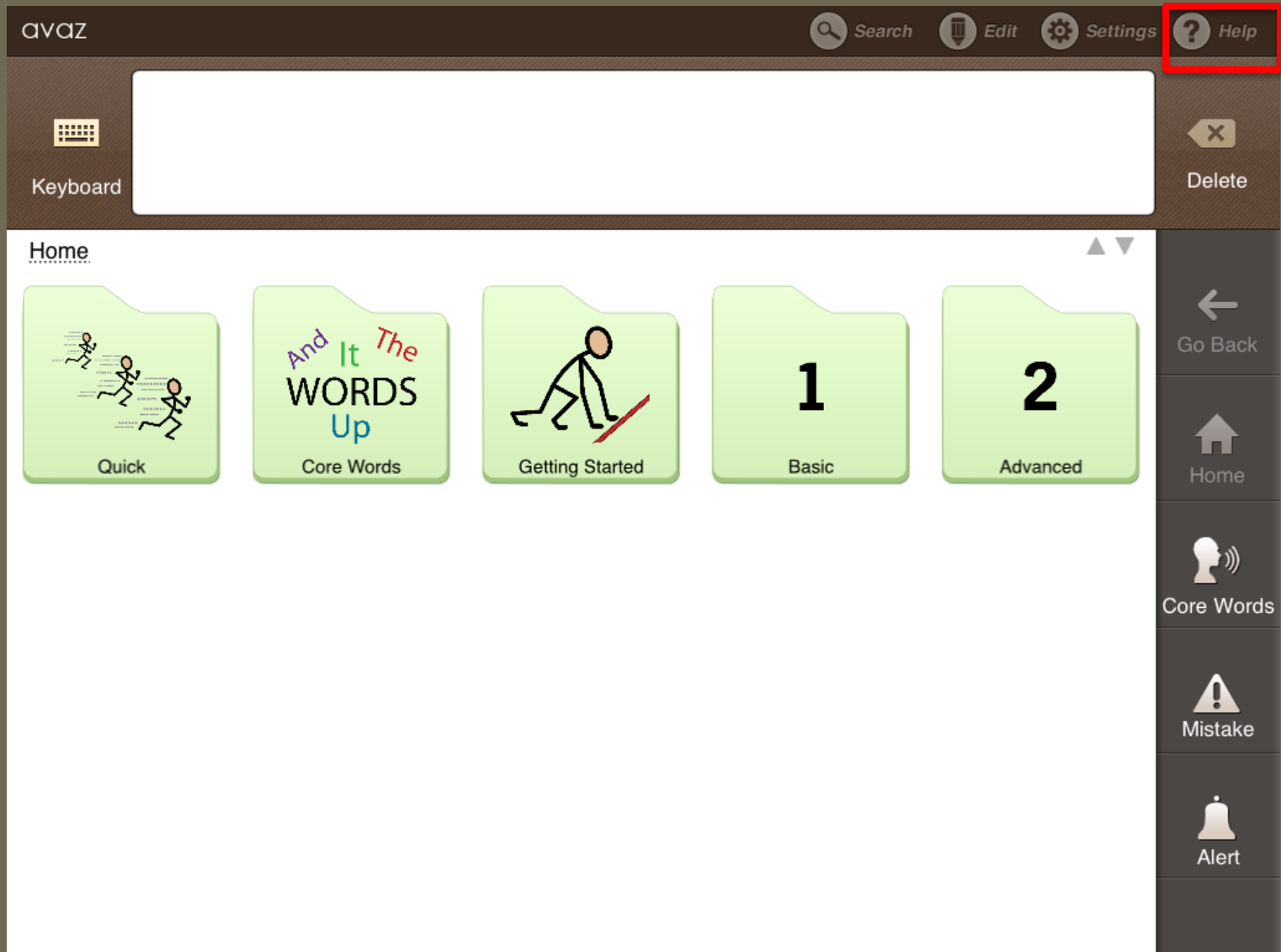
Ricerca vocabolario

The screenshot displays the AVAZ application interface. At the top, the AVAZ logo is on the left, and navigation icons for Search, Edit, Settings, and Help are on the right. The Search icon is highlighted with a red box. Below the top bar, there is a 'Keyboard' section and a 'Home' section with three folders: 'Quick', 'Core Words', and 'Getting'. A search modal is open, showing a search bar with the text 'color' and a list of search results. Each result includes an icon and a path to the word in the vocabulary.

AVAZ

Search Edit Settings Help

Keyboard

Home

Quick Core Words Getting

color

- Basic → indoor → color
- Basic → get dressed → color
- Advanced → describe → colors → color
- Advanced → categories → colors → color
- Advanced → my scenes → get dressed → choose color → color
- Quick → about me → my favorite color is
- Advanced → leisure-indoor → play board games → what color game...
- Advanced → leisure-indoor → do art and craft → color pencils
- Advanced → chat → about me → my favorite color is
- Advanced → categories → colors → favorite color
- Advanced → categories → stationery → color pencils
- Advanced → my scenes → at home → do homework → color pencils
- Advanced → my scenes → get dressed → choose color
- Advanced → my scenes → get dressed → choose color → favorite color

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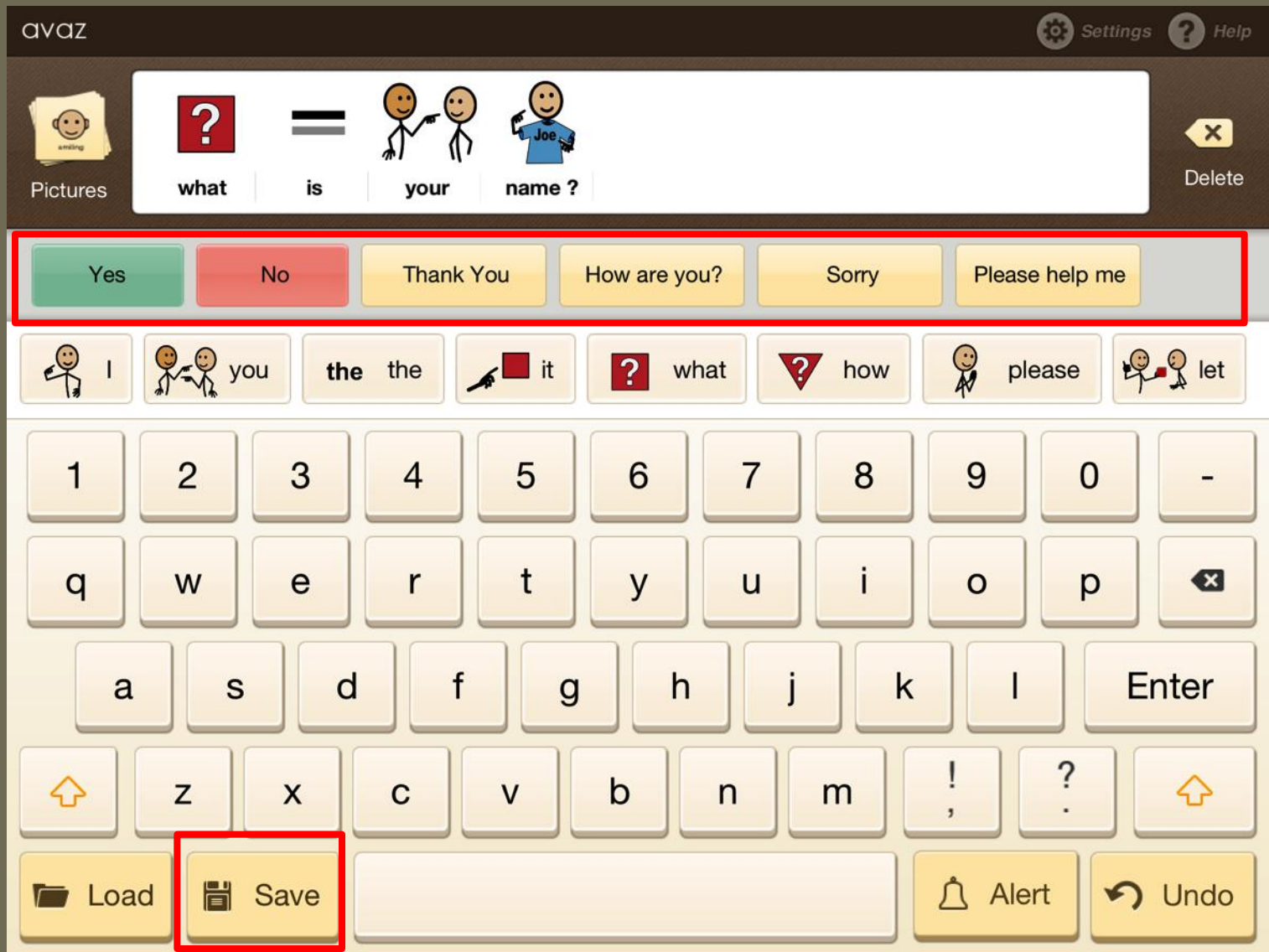
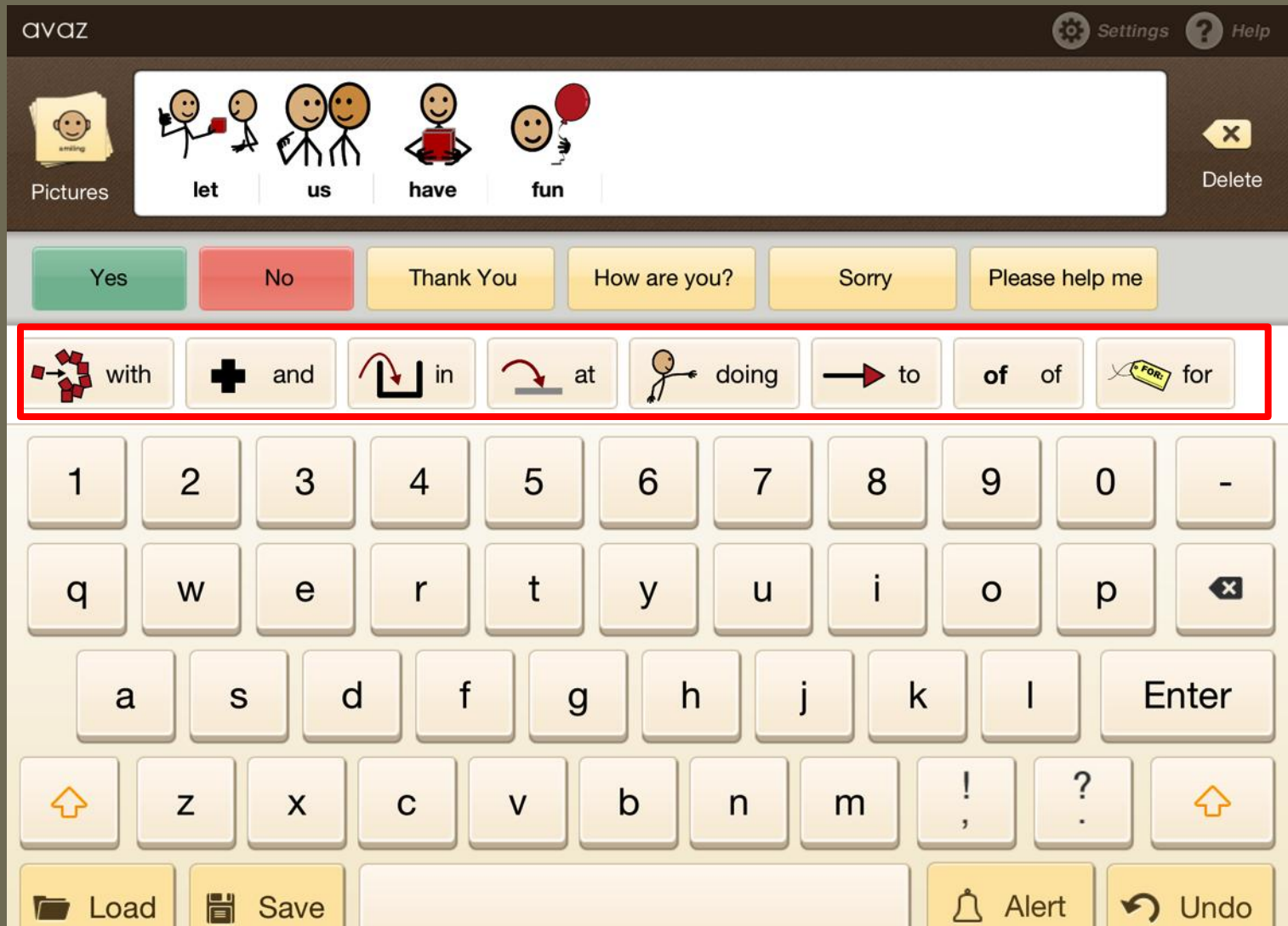
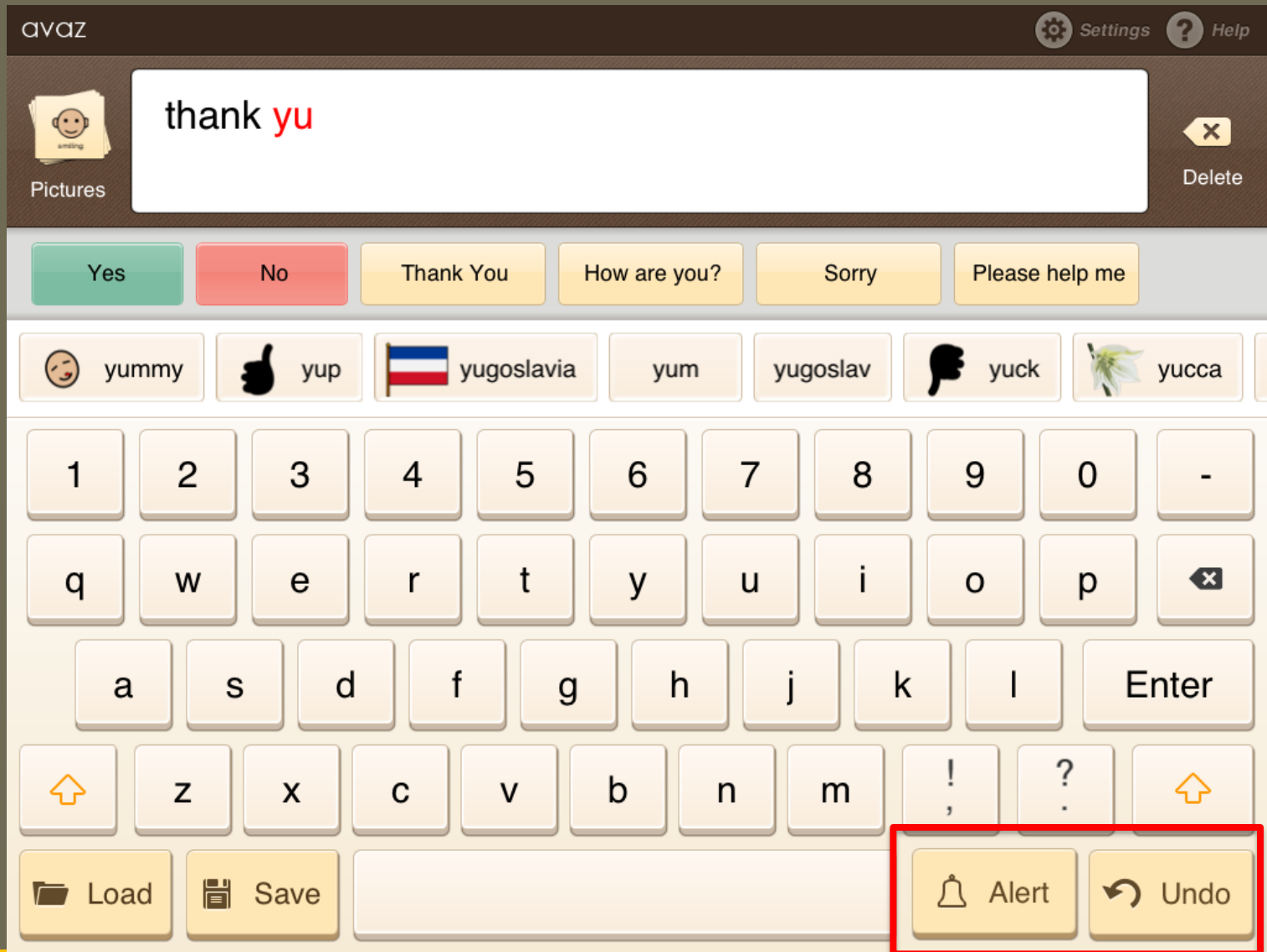
Carica

The screenshot shows the avaz app interface. At the top, there's a header with the 'avaz' logo, 'Settings', and 'Help' icons. Below the header is a message construction area with a 'Pictures' icon and a 'Delete' button. The message area contains the text 'good morning to you' with corresponding icons: a stick figure pointing up for 'good', a sun for 'morning', an arrow for 'to', and two stick figures for 'you'. Below the message area is a row of buttons: 'Yes', 'No', 'Thank You', 'How are you?', 'Sorry', and 'Please help me'. Underneath that is a row of message suggestions: 'good morning to you', 'you have any idea how to get there', 'please come in', and 'I think the people of ame'. The main part of the screen is a keyboard with several keys highlighted in red: '7', 'w', 'x', and the 'Load' button at the bottom left. The 'Load' button is a red folder icon with the text 'Load' next to it.

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The screenshot shows the avaz app interface. At the top, there's a navigation bar with 'avaz', 'Settings', and 'Help'. Below it, a 'Pictures' section shows icons for 'good', 'morning', 'to', and 'you'. A central prediction bar is highlighted with a red border, containing the following phrases: 'good morning to you', 'you have any idea how to get there', 'please come in', and 'I think the people of ame'. Below the prediction bar is a virtual keyboard with several keys highlighted in red: '7', 'w', and 'x'. At the bottom, there are buttons for 'Load', 'Save', 'Alert', and 'Undo'.

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Home → Advanced → people

I call don't is like with

name what where who who is that? my family

my pets school people more people helpers pronouns professions

describe nice to me not nice to me characters someone else

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Riorganizza elementi

The screenshot shows the AVAZ application interface. At the top, there is a navigation bar with the AVAZ logo, a search icon, and buttons for 'Search', 'Edit', 'Settings', and 'Help'. Below the navigation bar is a large white text input field with a plus icon on the left labeled 'Add new' and a change icon on the right labeled 'Change'. The main content area displays a breadcrumb trail: 'Home → Advanced → people → my family'. Below the trail is a grid of 24 icons representing various family and social relationships, each with a label: 'my', 'family', 'mom', 'dad', 'brother', 'sister', 'uncle', 'aunt', 'babysitter', 'caregiver', 'cousin', 'grandfather', 'grandmother', 'grandchild', 'grandparents', 'neighbor', 'younger', 'older', 'pet', 'is home', and 'is at work'. A 'your' icon is also visible, overlapping the 'uncle' and 'aunt' icons. On the right side, there is a vertical sidebar with five action buttons: 'Go Back', 'Copy', 'Delete', 'Enable', and 'Disable'.

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The screenshot shows the avaz application interface. At the top, there is a navigation bar with the logo 'avaz' on the left and icons for 'Search', 'Edit', 'Settings', and 'Help' on the right. Below the navigation bar, there are two buttons: 'Add new' (with a plus sign icon) on the left and 'Change' (with a document icon) on the right, both highlighted with red boxes. The main content area displays a grid of icons representing various categories and messages. A modal dialog is open in the center, asking 'What do you want to create?' with two options: 'New message' and 'New category'. The grid includes icons for 'I', 'call', 'like', 'with', 'name', 'what', 'where', 'who', 'who is that?', 'my family', 'my pets', 'school people', 'more people', 'helpers', 'pronouns', 'professions', 'describe', 'nice to me', 'not nice to me', 'characters', and 'someone else'. On the right side, there is a vertical sidebar with buttons for 'Go Back', 'Copy', 'Delete', 'Enable', and 'Disable'.

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The screenshot shows the avaz app interface with a dialog box for editing a symbol. The dialog box has a title bar with 'Cancel' and 'Done' buttons. The main content area shows a preview of a family icon with the caption 'my family'. Below the preview are several options for customizing the symbol:

- Caption:** A text field containing 'my family'.
- Show in message box:** A toggle switch currently set to 'OFF'.
- Color:** A palette of colored circles (orange, green, purple, yellow, blue, grey, brown) with a close button (X).
- Find symbol:** A search bar.
- Take Photo / Choose Photo:** Two buttons for selecting an image.
- Speak as:** A text field with a speaker icon, a red circle icon, and a trash icon.

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AVAZ Search Edit Settings Help

Cancel Done

people_symbolstix/family/family_2.png

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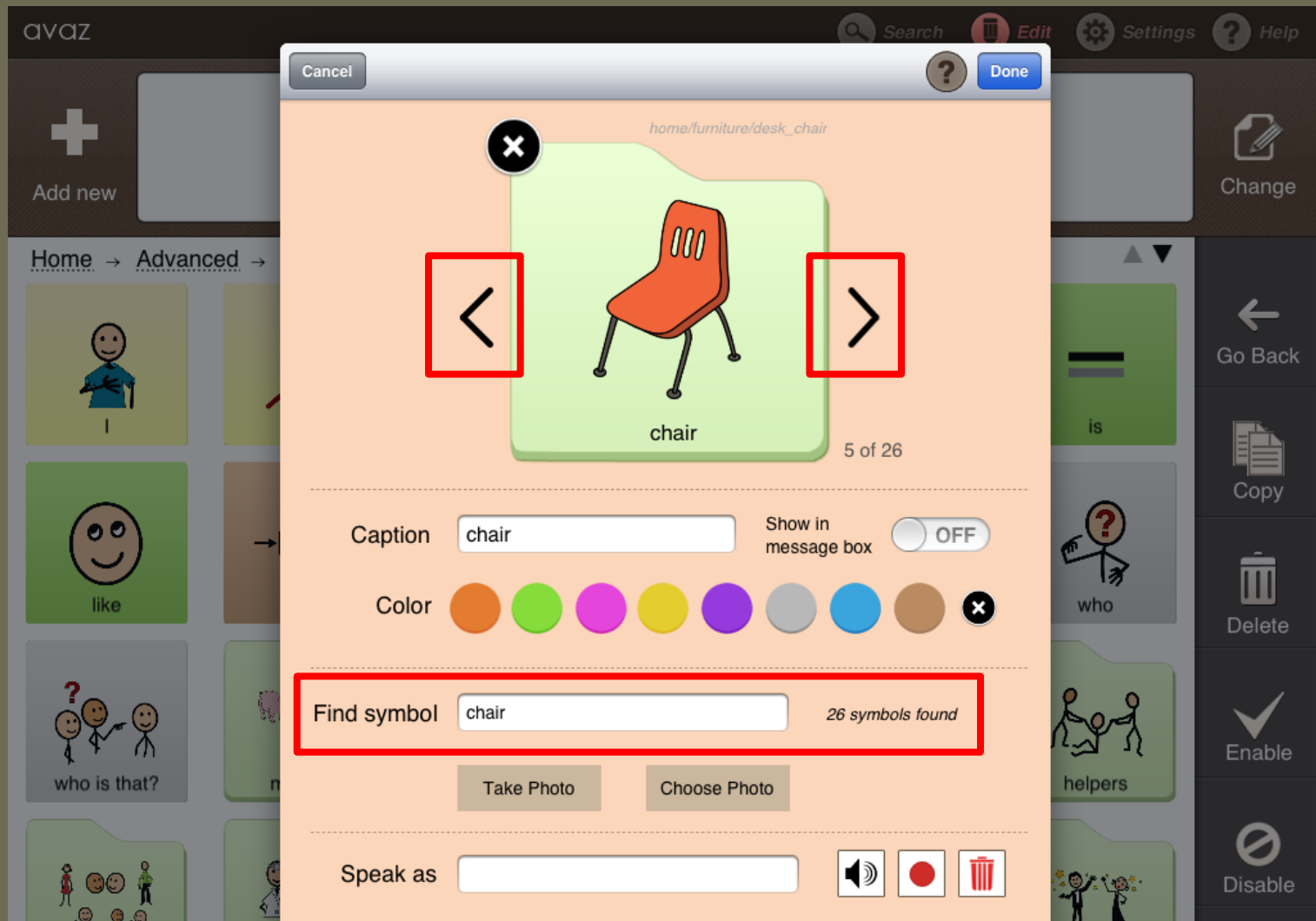
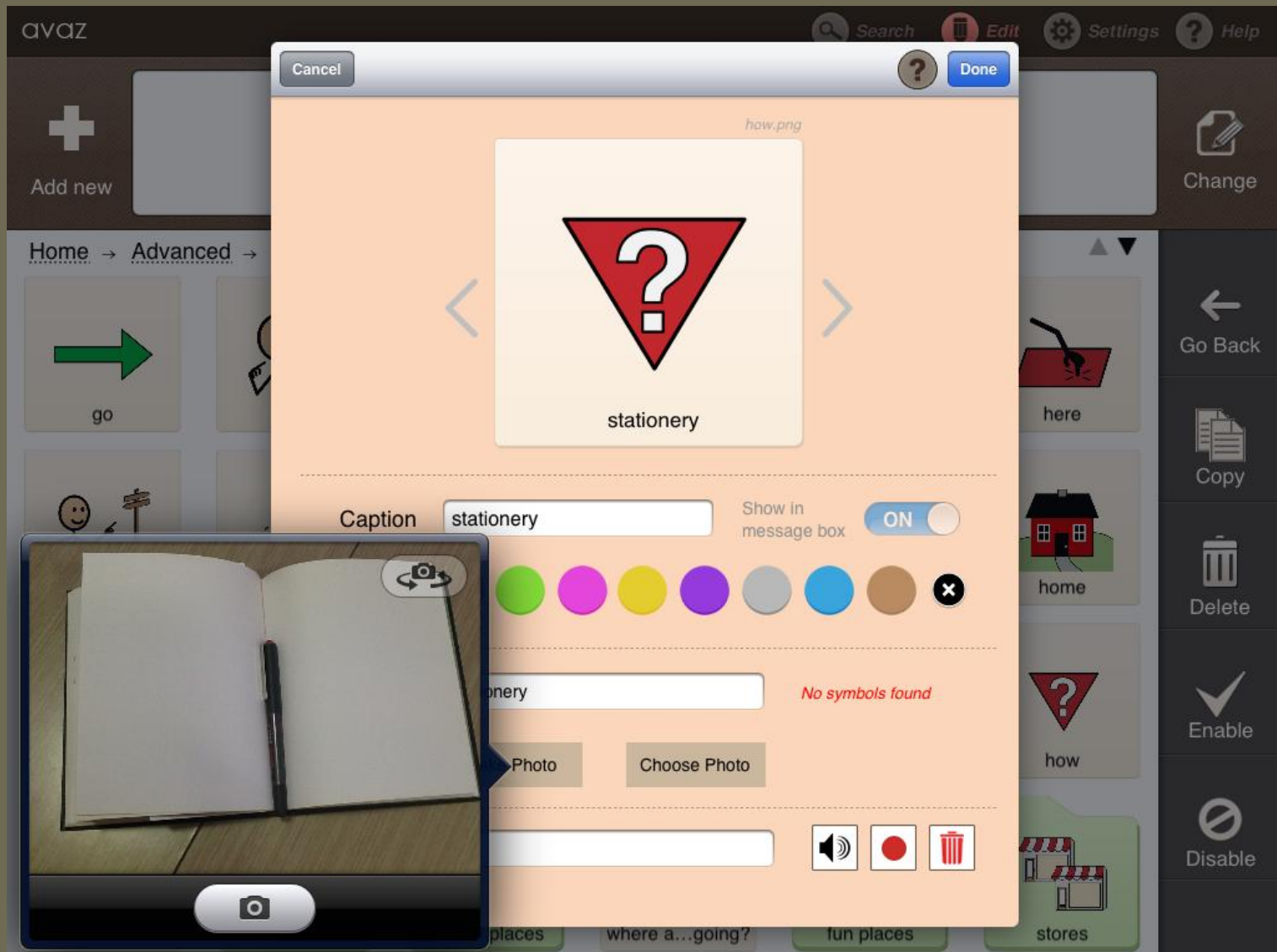
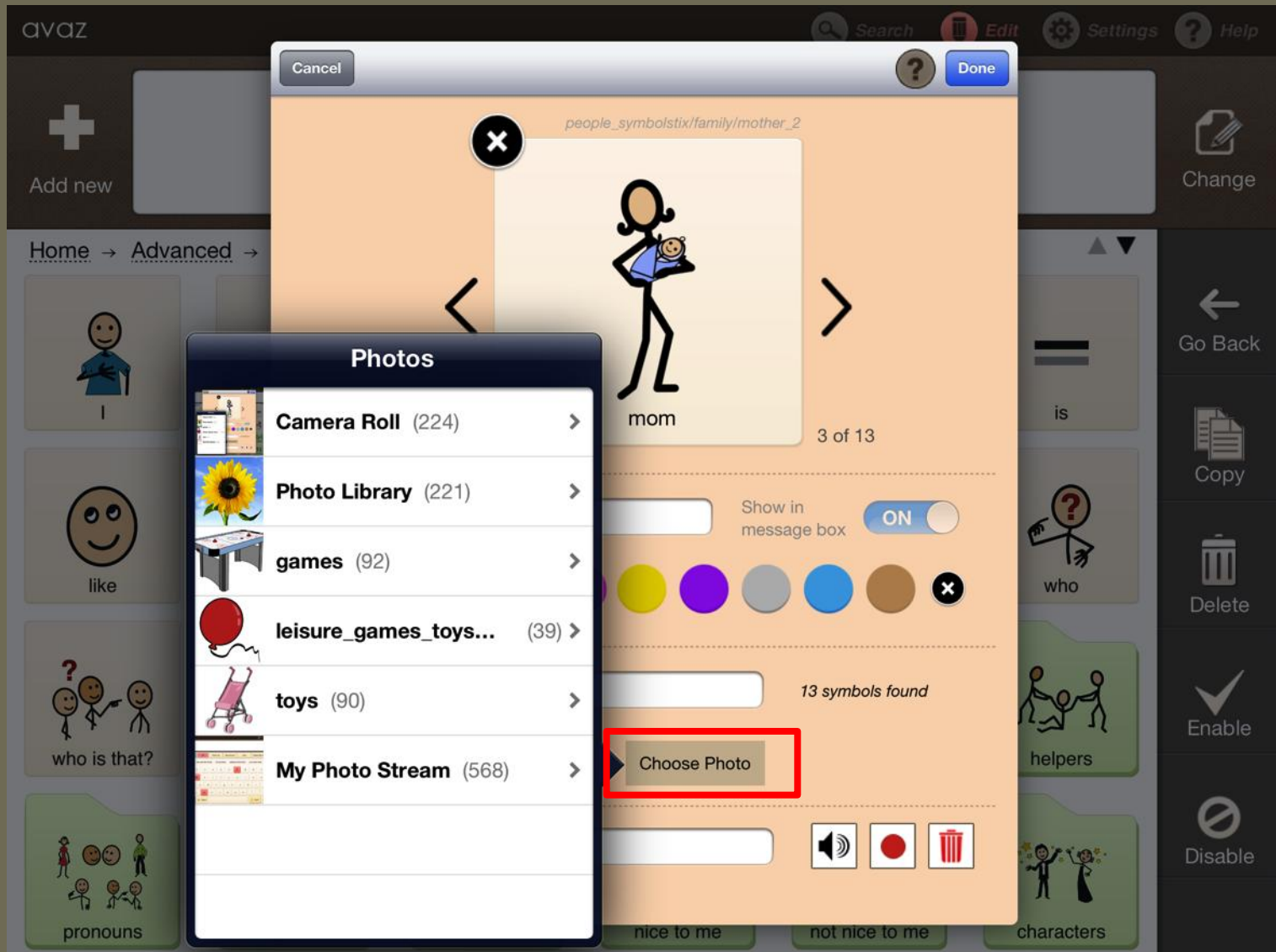
Avaz: Impostazioni

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The screenshot shows the Avaz application interface. At the top, there is a navigation bar with the 'avaz' logo on the left and icons for Search, Edit, Settings, and Help on the right. Below the navigation bar, there is a 'Keyboard' section with a keyboard icon. The main content area is titled 'Home' and features four green folder-like icons: 'Quick' (with a stick figure running), 'Core Words' (with the text 'And It The WORDS Up'), 'Getting Started' (with a stick figure pushing a red bar), and 'Basic' (with the number '1'). A settings menu is overlaid on the right side of the screen. The menu is titled 'General' and has two tabs: 'Keyboard' and 'Pictures'. The 'Pictures' tab is selected. The settings menu includes the following options:

- Start with: Keyboard (selected), Pictures
- Share on social media >
- Backup & Restore >
- Track session: Start (selected), Stop
- Get Avaz tips & more >
- Picture settings (Appearance)
 - Message box: ON (selected)
 - Pictures in msg box: ON (selected)
 - Pictures per screen >
 - Caption size >
 - High contrast: OFF
 - Color coding >
- Picture settings (Behavior)
 - Zoom on select >

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
Condividi da Avaz

The screenshot displays the Avaz application interface. At the top, the word "avaz" is on the left, and navigation icons for "Search", "Edit", "Settings", and "Help" are on the right. Below this is a horizontal menu with icons for "Keyboard", a person, a smiley face, an arrow, a hand holding a red card, a tablet, and a Facebook icon. The text "I like to use Avaz" is positioned below these icons. The Facebook icon is highlighted with a red square. To the right of the Facebook icon is a "Delete" button with an 'X' icon. Below the menu is a "Home" section with five green folder-like icons: "Quick" (with a running stick figure), "Core Words" (with the text "And It The WORDS Up"), "Getting Started" (with a stick figure on a ramp), "Basic" (with the number "1"), and "Advanced" (with the number "2"). On the right side of the screen is a vertical sidebar with icons for "Go Back", "Home", "Quick", "Mistake", and "Alert".

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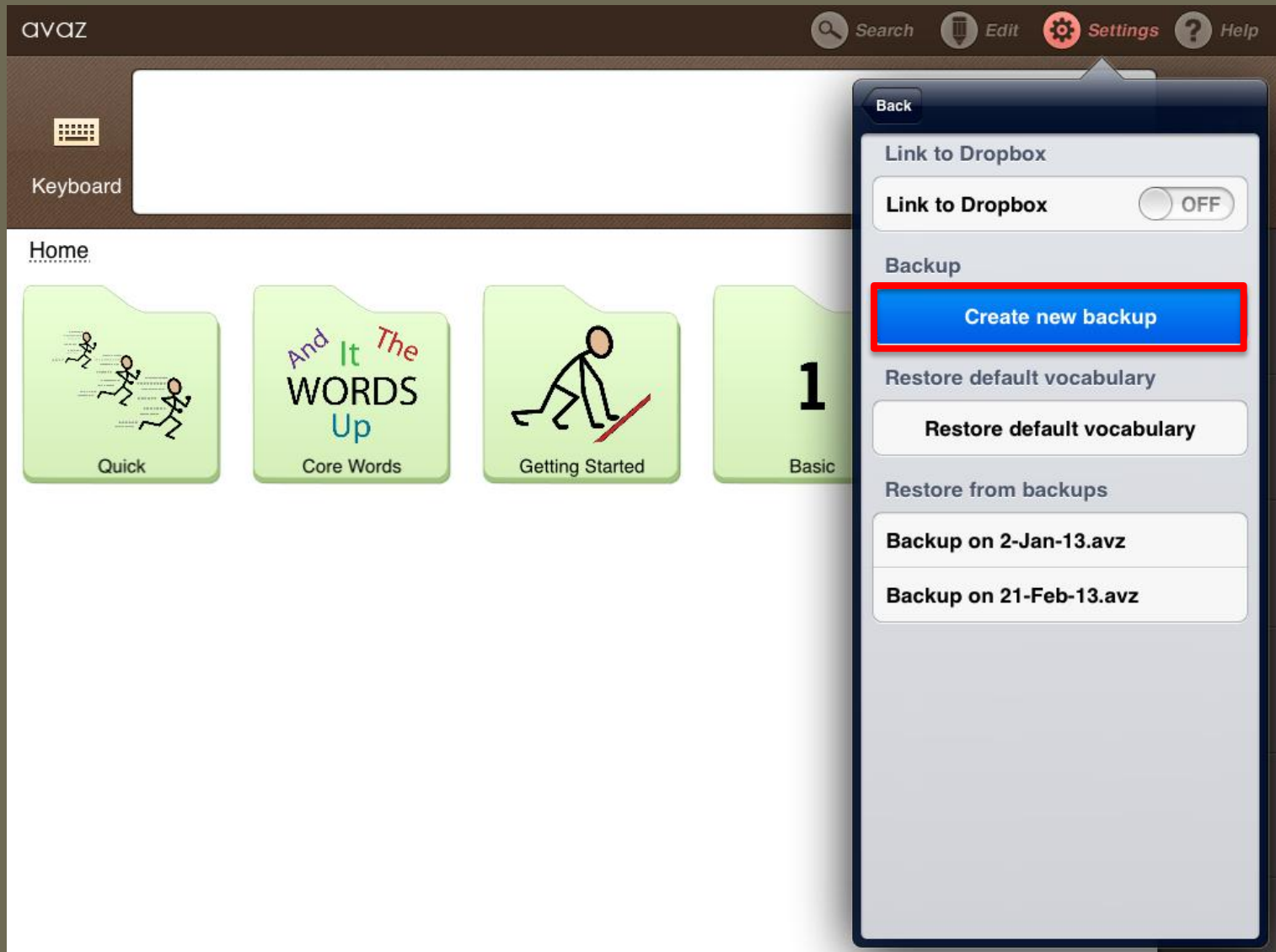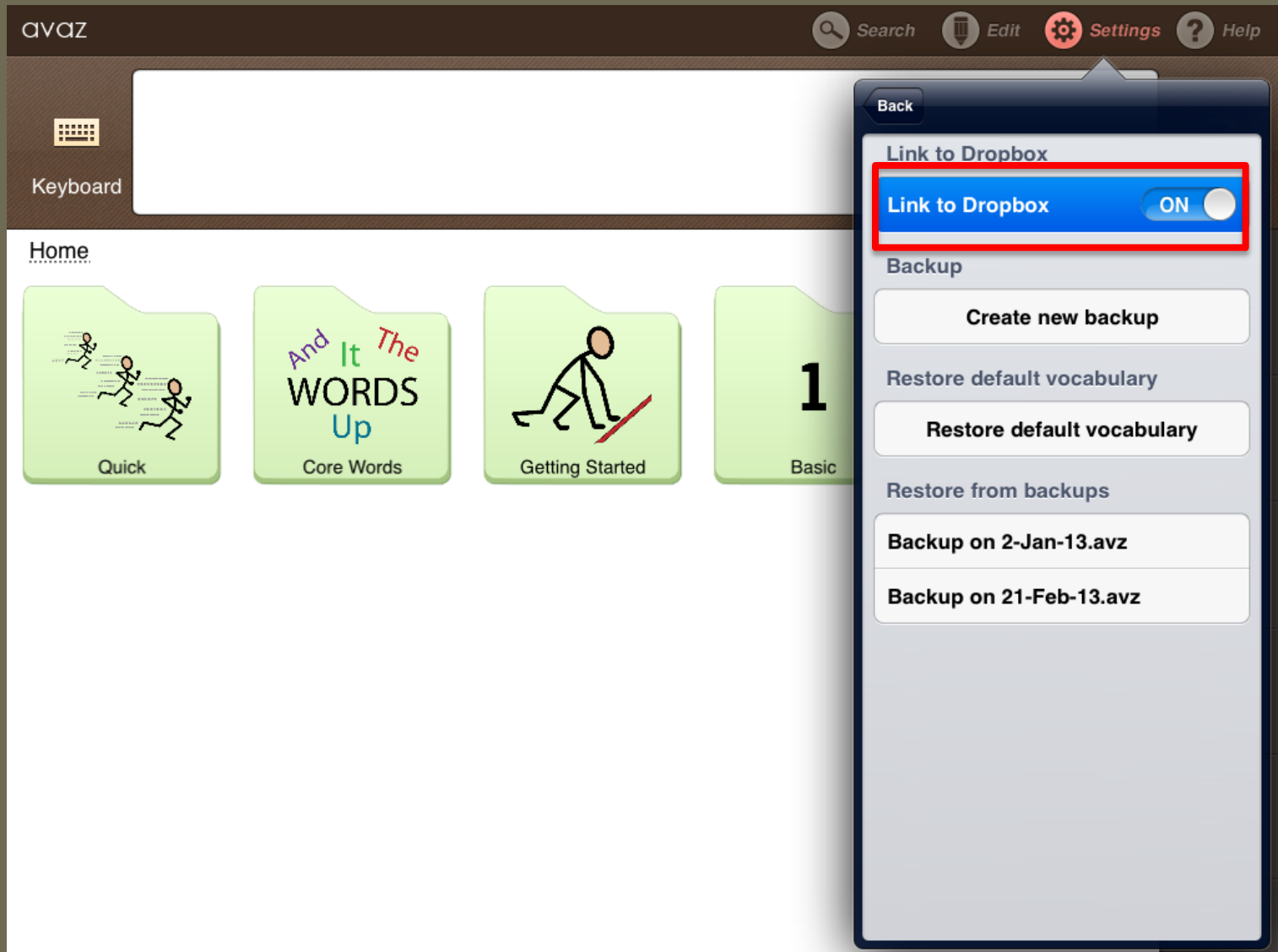
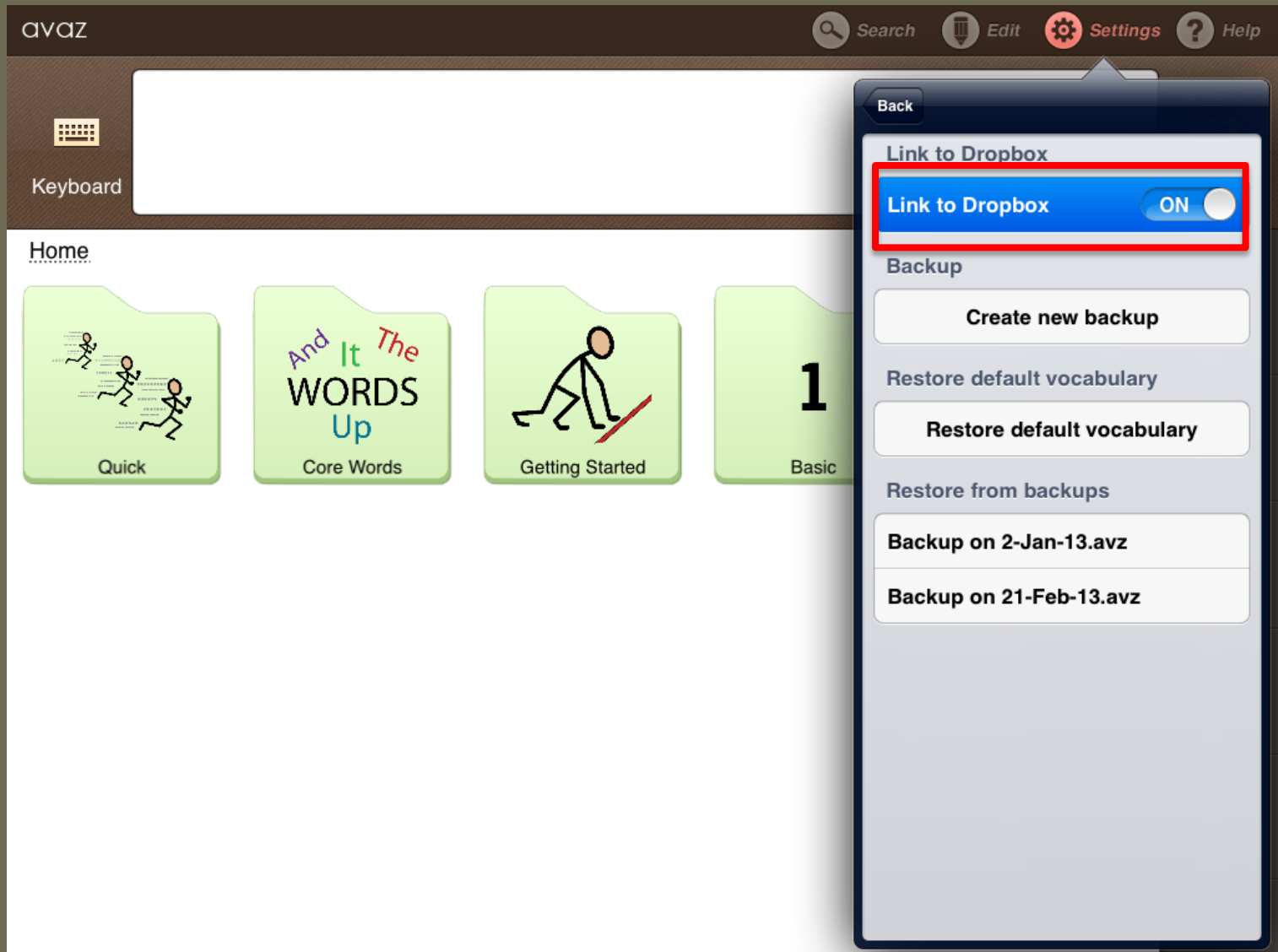
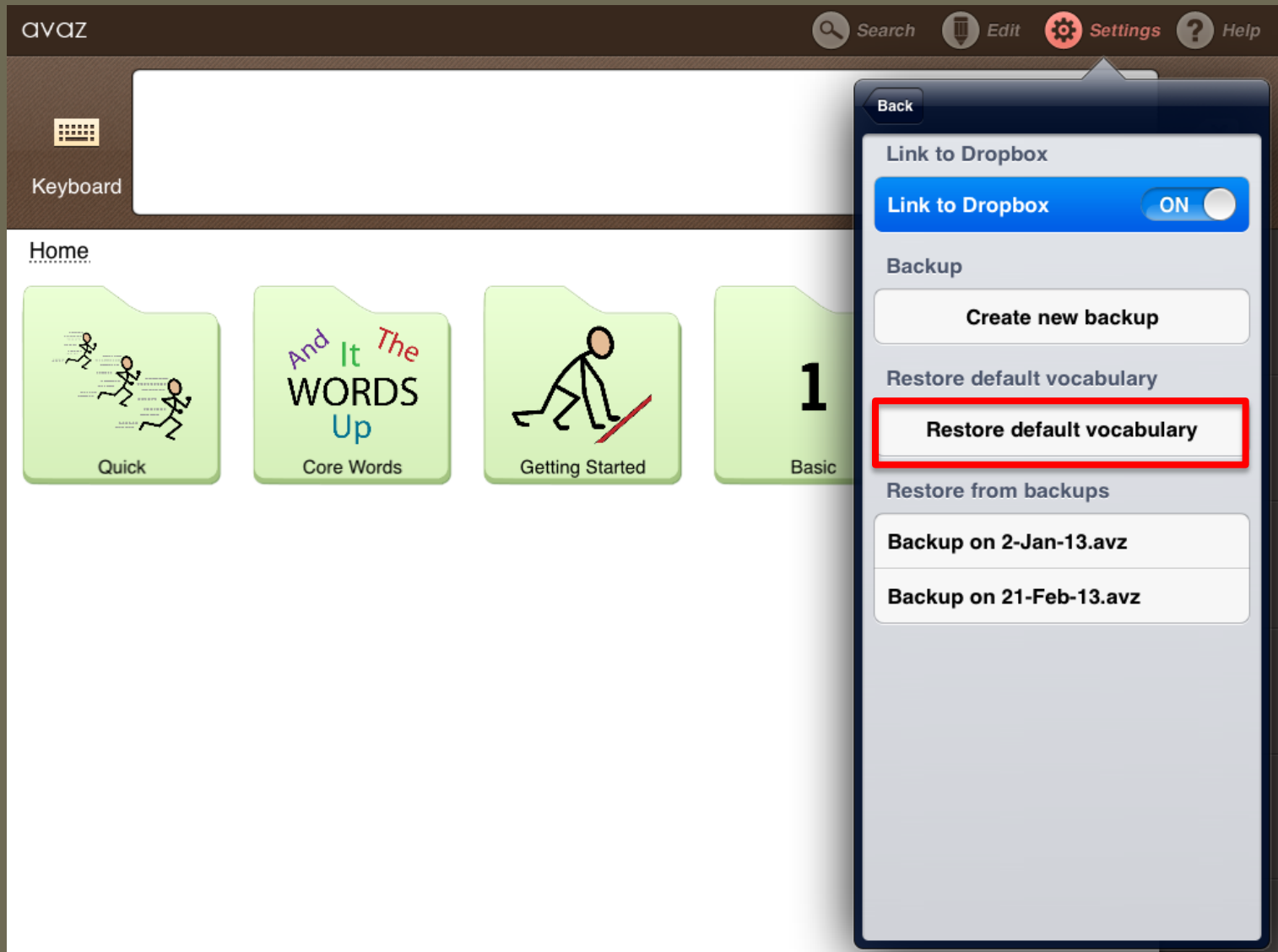
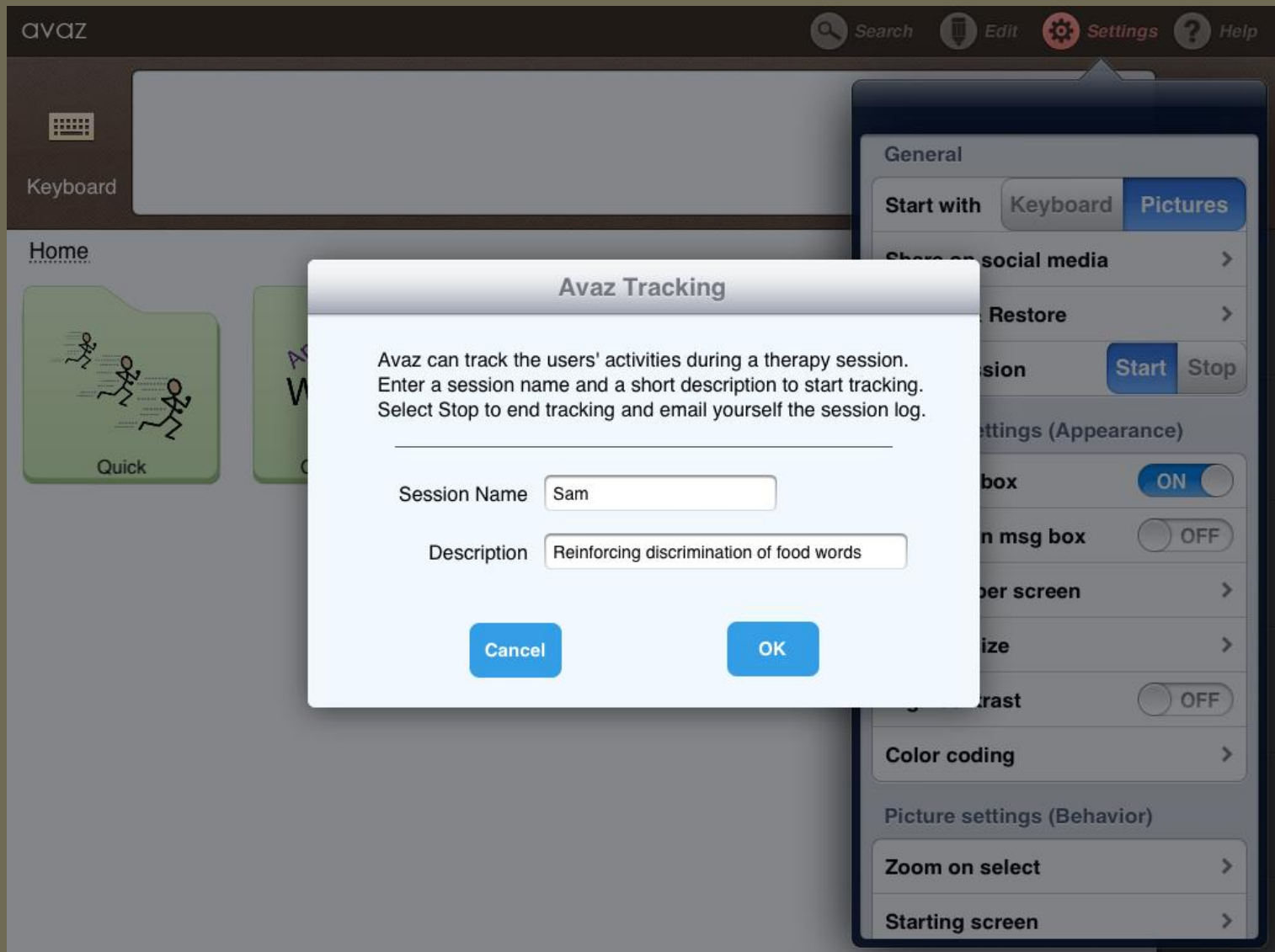
The screenshot displays the avAZ application interface. At the top, there is a navigation bar with the 'avAZ' logo on the left and icons for 'Search', 'Edit', 'Settings', and 'Help' on the right. Below the navigation bar, a 'Keyboard' section is visible. The main content area is titled 'Home' and features four large, light green buttons: 'Quick' (with an icon of three running stick figures), 'Core Words' (with the text 'And It The WORDS Up'), 'Getting Started' (with an icon of a stick figure on a red ramp), and 'Basic' (with a large number '1'). A settings menu is overlaid on the right side of the screen, outlined in red. This menu is divided into two sections: 'Picture settings (Appearance)' and 'Picture settings (Behavior)'. The 'Appearance' section includes options for 'Message box' (ON), 'Pictures in msg box' (OFF), 'Pictures per screen', 'Caption size', 'High contrast' (OFF), and 'Color coding'. The 'Behavior' section includes options for 'Zoom on select', 'Starting screen', 'Auto-home each time' (OFF), 'Rearrange on disable' (ON), 'Page up & down keys' (OFF), and 'Quick/Core Words button'.

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avaz Search Edit Settings Help

Home → Advanced → indoor → play with toys

 toy	 baby doll	 ball	 ball pit	 balloon
 barbie	 bat	 beads	 bean bag	 bounce on ball
 bubble wand	 bubbles	 bucket	 cars	 doll
 dollhouse	 jump rope	 Playmobil	 pull toy	 puppet

Keyboard
Go Back
Home
Quick
Mistake
Alert

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avaz Search Edit Settings Help

Keyboard I want soup Delete

Home → Advanced → food → lunch and dinner ▲ ▼

salad

soup

Go Back Home Quick Mistake Alert

The screenshot shows the Avaz application interface. At the top, there is a dark header with the 'avaz' logo and navigation icons for Search, Edit, Settings, and Help. Below the header is a white input area containing three icons: a person, a person pointing at a red square, and a bowl of soup. Below these icons are the words 'I', 'want', and 'soup'. To the right of this area is a 'Delete' button. Below the input area is a breadcrumb trail: 'Home → Advanced → food → lunch and dinner'. The main content area displays two large, light-colored rectangular cards. The left card shows a bowl of salad with the word 'salad' below it. The right card shows a bowl of soup with the word 'soup' below it. On the right side of the screen, there is a vertical sidebar with several navigation buttons: 'Go Back' (left arrow), 'Home' (house icon), 'Quick' (person with sound waves), 'Mistake' (warning triangle), and 'Alert' (bell icon).

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
The screenshot shows the Avaz application interface. At the top, there is a navigation bar with the 'avaz' logo, a search icon, and buttons for 'Edit', 'Settings', and 'Help'. Below the navigation bar is a 'Keyboard' section with a keyboard icon. The main content area is titled 'Home → Advanced' and displays a grid of 16 word cards. Each card contains an icon and a word: 'I' (person), 'you' (two people), 'that' (hand pointing to a red square), 'do' (person with arm out), 'is' (equals sign), 'like' (smiley face), 'want' (person with hand over a red square), 'to' (arrow), 'not' (red X), 'what' (red square with question mark), 'food' (plate of food), and 'indoor' (people sitting at a table). A settings menu is open on the right side, titled 'Back', and lists five options: 'No caption (only image)', 'Small', 'Medium', 'Large' (which is selected with a blue checkmark), and 'Only text (no image)'.

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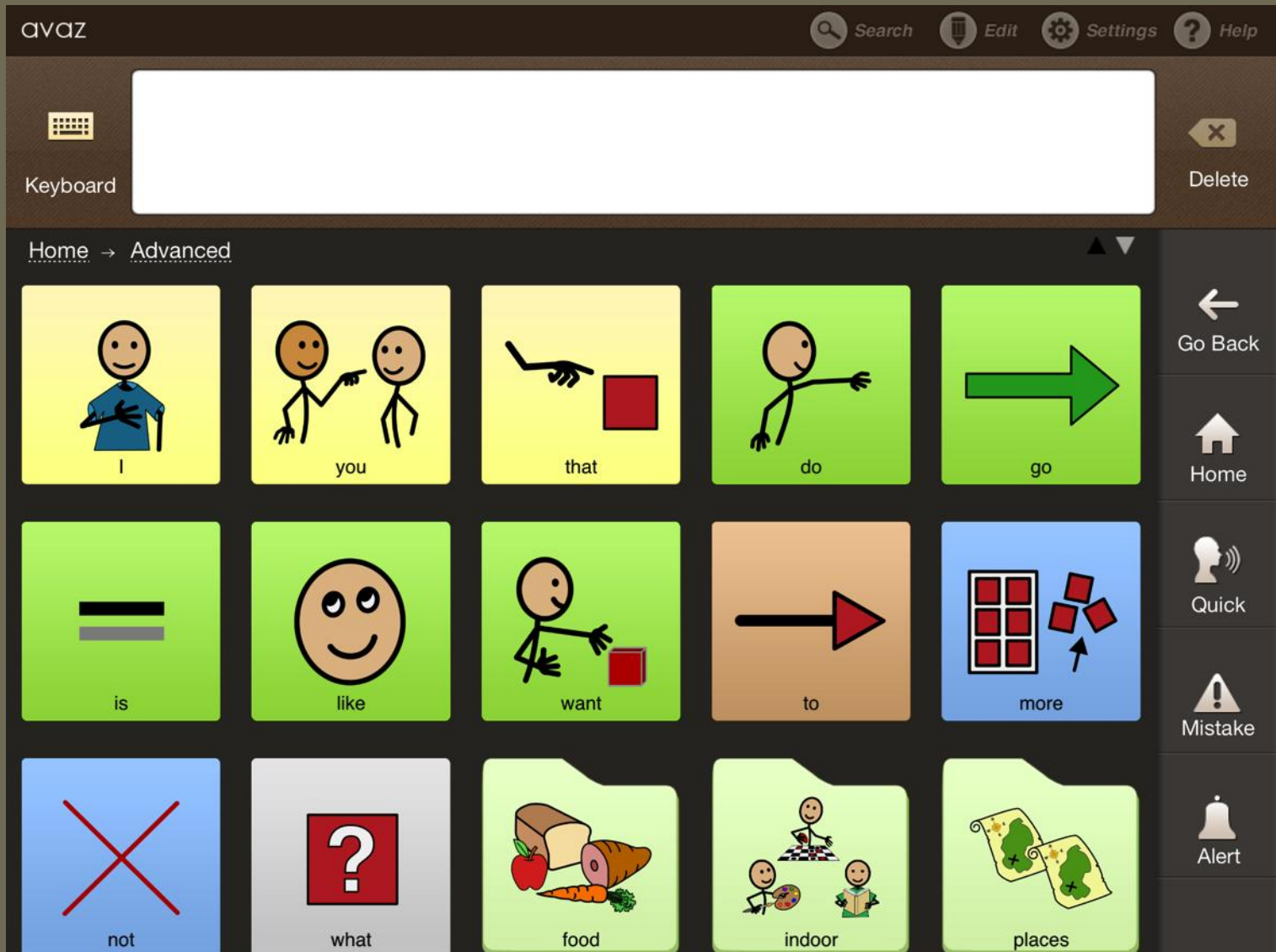
The screenshot displays the Avaz application interface. At the top, the 'avaz' logo is on the left, and navigation icons for 'Search', 'Edit', 'Settings', and 'Help' are on the right. Below this is a 'Keyboard' section with icons for 'play' (two stick figures) and 'drums' (a red drum). A 'Delete' button with an 'X' icon is on the right. The main content area shows a breadcrumb trail: 'Home -> Advanced -> indoor -> listen to music'. A grid of icons represents various musical instruments: 'band', 'drums' (enlarged), 'clarinet', 'cymbal', 'flute', and 'guitar'. Other visible icons include an electric guitar, harmonica, keyboard, maracas, and piano. On the right side, a vertical navigation bar contains icons for 'Go Back', 'Home', 'Quick', 'Mistake', and 'Alert'.

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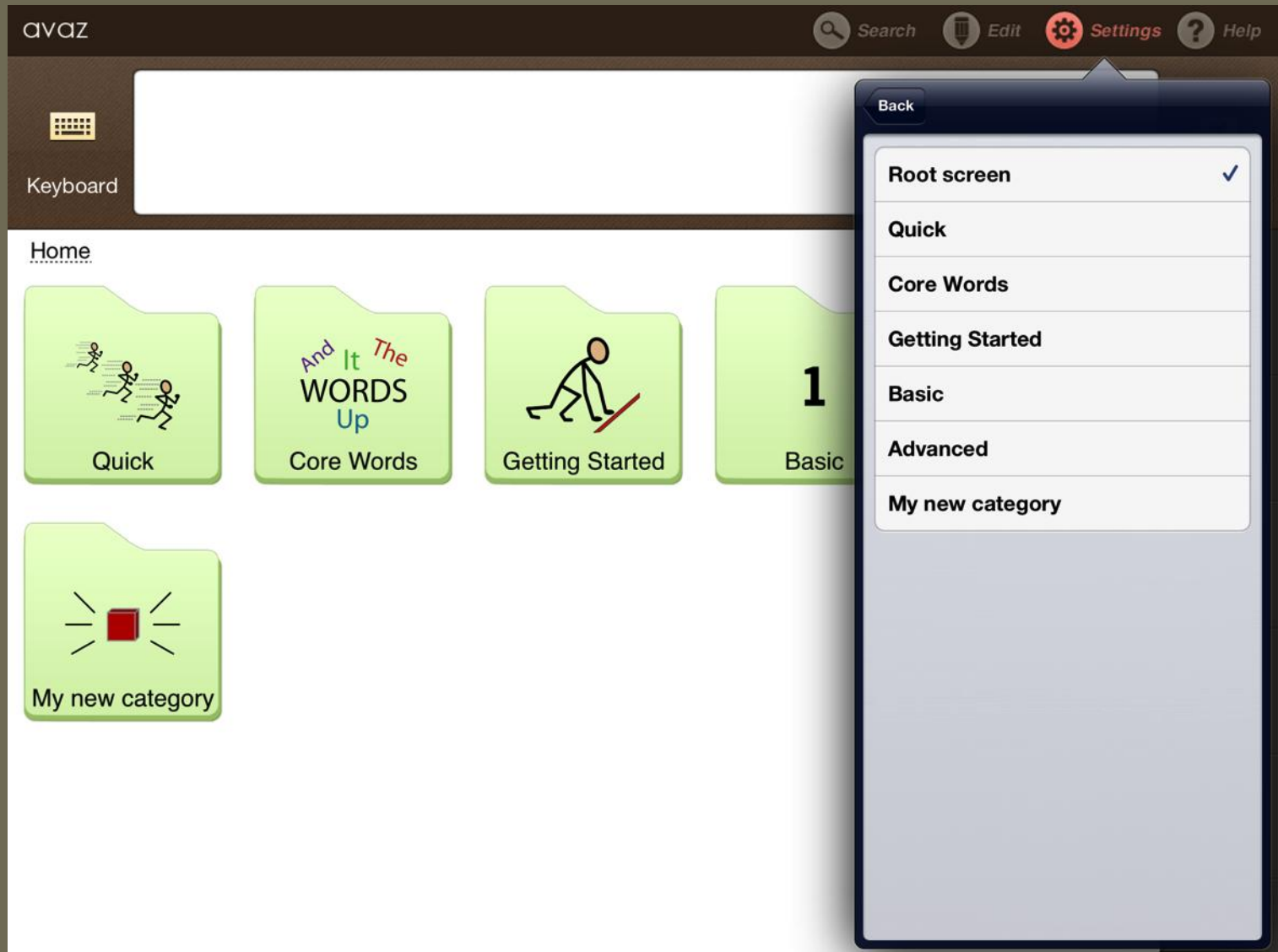
The screenshot displays the Avaz application interface. At the top, there is a navigation bar with the 'Avaz' logo, a search icon, and buttons for 'Edit', 'Settings', and 'Help'. Below the navigation bar, a 'Keyboard' section is visible. The main content area is titled 'Home' and features four green folder-style icons: 'Quick' (with a stick figure), 'Core Words' (with the text 'And It The WORDS Up'), 'Getting Started' (with a stick figure on a ramp), and 'Basic' (with the number '1'). A settings menu is open on the right side, with the 'Keyboard settings' section highlighted by a red box. This section includes a 'Layout' toggle set to 'QWE' (with 'ABC' also visible) and a 'Predict pictures' toggle set to 'ON'. Other settings sections visible include 'Audio settings' with 'Voice', 'Speed', 'Speak as you type', and 'Speak action keys' (all 'ON'), and buttons for 'Change settings password' and 'Avaz support'. At the bottom of the settings menu, it states 'You are running Avaz version 2.0'.

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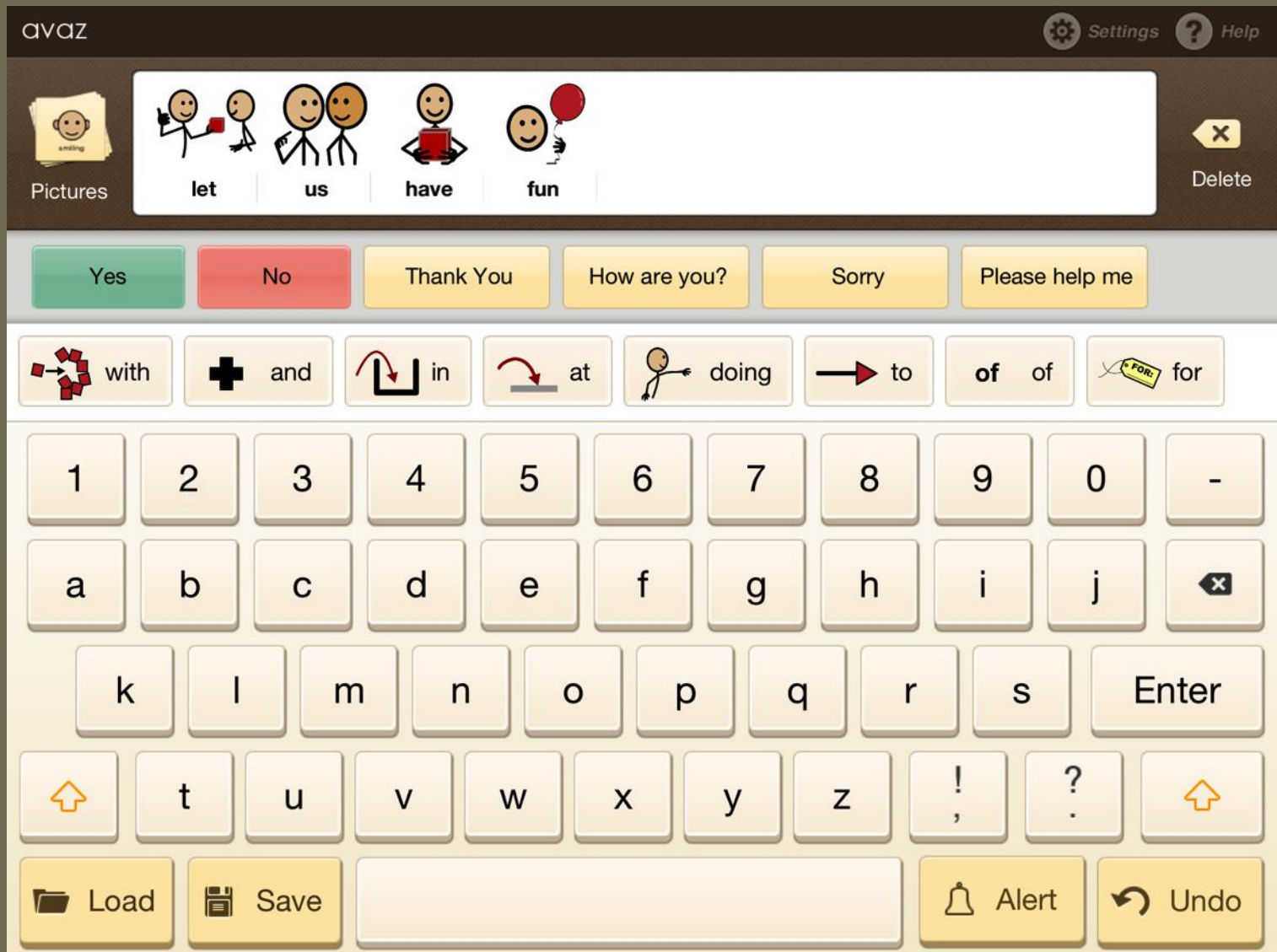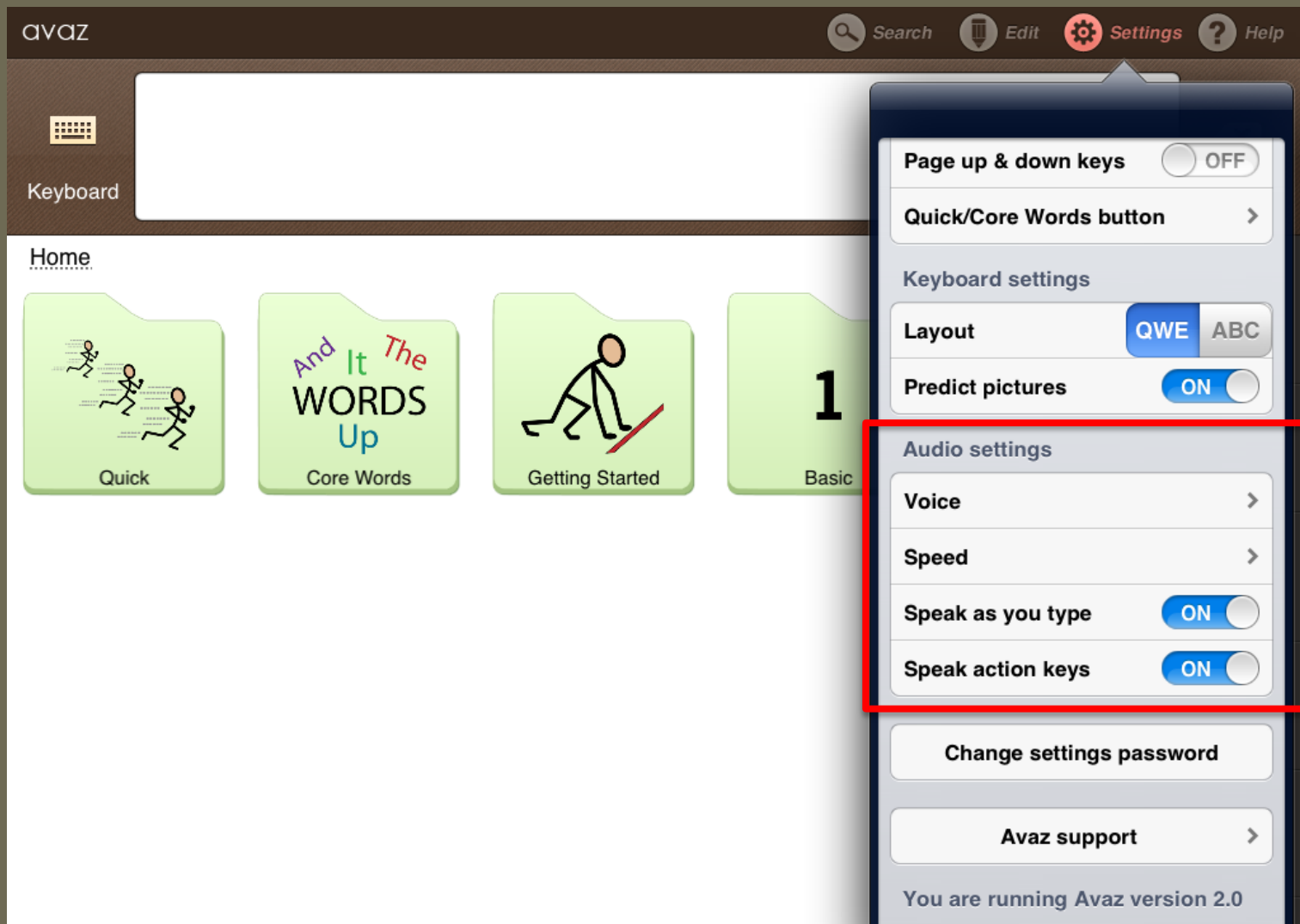
The screenshot displays the Avaz application interface. At the top, the 'avaz' logo is on the left, and navigation icons for 'Search', 'Edit', 'Settings', and 'Help' are on the right. Below the logo is a 'Keyboard' icon and a text input field. The main content area is titled 'Home → Advanced' and features a grid of word cards. Each card contains an illustration and a word: 'I' (stick figure), 'you' (two stick figures), 'that' (hand pointing to a red square), 'do' (stick figure), 'is' (equals sign), 'like' (smiling face), 'want' (stick figure with a red box), 'to' (arrow), 'food' (apple, carrot, bread), 'indoor' (stick figures playing board games), 'places' (maps), and 'people' (group of faces). A dark blue menu is open on the right, titled 'Back', listing voice options: 'Ryan (US Male)', 'Laura (US Female)' (selected with a checkmark), 'Kenny (US Child)', 'Nelly (US Child)', 'Deepa (Indian Female)', 'Graham (British Male)', and 'Lucy (British Female)'.

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